



Catalogue 2021

For all your laryngectomy,
tracheostomy and jaw
rehabilitation supplies



Contents

Laryngectomy

Breathing aids for laryngectomy

Information and care tips	10 - 17
Provox Life introduction	18 - 23
Provox Life HMEs	24 - 26
Provox FreeHands FlexiVoice	27
Provox FreeHands FlexiVoice Support	27
Provox Life Adhesives and Baseplates	28 - 30
Provox Life Tubes and Buttons	31 - 32
Provox Life Shower	33
Provox Holders and Clips	33
Provox HMEs	36 - 37
Provox Adhesives and Baseplates	38 - 39
Provox Tubes and Buttons	40 - 41
SpiroTech System	42
Blomsinger System	43 - 44

Accessories for laryngectomy

Tube Swabs and Brushes	46
Voice Prosthesis Brushes	47
Shower Protectors	47
Stoma Protectors	48 - 49
Neckbands	50 - 51
Foam Covers	52
Lubricants	52
Skin Wipes and Creams	53
Silicone Adhesive	54
Irrigation Fluid	54
Gauze Swabs	55 - 56

Support and Services

Customer Care Team	58 - 59
Nursing Service	60 - 61
MyLife App	62 - 63
MyVoice Events	64 - 65

Tracheostomy

Breathing aids for tracheostomy

Information and care tips	66 - 69
Freevent XtraCare	72
Freevent DualCare	73
TrachPhone	74
Tubes	74

Accessories for tracheostomy

Tube Swabs and Brushes	76 - 77
Shower Protectors	77
Stoma Protectors	78 - 79
Neckbands	80 - 81
Foam Covers	82
Lubricants	82
Skin Wipes and Creams	83
Irrigation Fluid	83
Gauze Swabs	84
Dressings	85 - 86

Jaw Rehabilitation

TheraBite	88
-----------	----



Product shown: Provox Life Energy

Welcome to Atos Care

At Atos Care, our mission is to help make life easier for people living with a neck stoma. We understand that having a laryngectomy or tracheostomy can be life changing, and we want to be here to support you every step of the way.



In this catalogue you will find many of the products we supply, including a brand new range of products - Provox Life - which are launching in March 2021.

You may have noticed that we recently changed our name from Countrywide Supplies to Atos Care. You can still expect the same helpful team and the same great service. Our name change reflects the comprehensive range of care and support services we offer, as well as convenient delivery of your prescription medical supplies directly to your door.


If you can't find what you're looking for, please visit our website, which has a wide range of products listed, or contact our helpful Customer Care Team, who can help you find the products you are looking for if they are available on prescription. We supply products made in-house by Atos (the Provox Life, Provox and FreeVent ranges) and also from a range of other manufacturers, to give you the best choice and selection.

We are here to give you a helping hand, whatever you may need. If you have any questions, queries or feedback, please do not hesitate to get in touch.

Best wishes,
Lizz Summers
Head of Customer Care
Atos Care



www.atos-care.co.uk

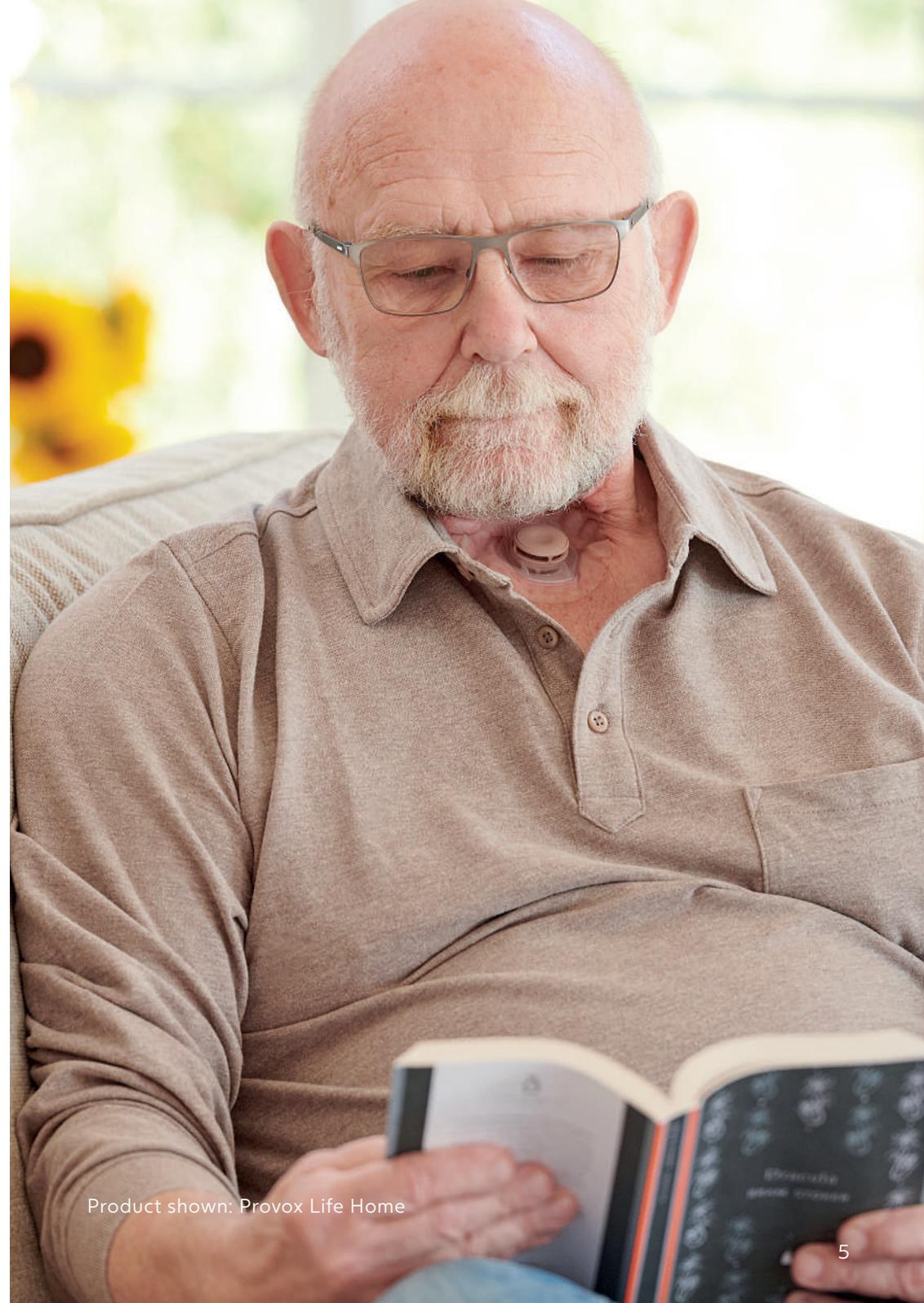

Product shown: Provox Life Home

The Atos Circle of Care

We understand that after your operation, you need more than just products to help you live your life fully. Trust, time and care are more important than ever. The Atos Circle of Care is our comprehensive support service to help you.

Best Start: Get off to the best start in life after a laryngectomy or tracheostomy.

The Best Start programme helps you transition from hospital to home, including adjusting to your new condition, understanding your new products and getting in to good routines.

- Enhanced support for your first six months from our Welcome Team of CQC Registered Nurses
- A welcome pack and a welcome call
- Practical tools and equipment to make life easier, including a complimentary care bag containing a range of helpful items
- Regular liaison with clinicians for joined up care



Connection Hub: Helping you stay connected right from the beginning of your journey.

Get comfortable with your new life and stay connected to those who know and understand.

- A dedicated Customer Care Representative
- Personalised service - you choose how and when you hear from us
- CQC Registered Nurses to support you in your daily routines, in close partnership and communication with clinicians
- Educational events in the community for people with similar experiences
- Atos MyLife App to provide inspiration and information on living well with a neck stoma



Care Delivered: Bringing the right products and care your way.

The right products for your needs, right when you need them.

- Easy and convenient ordering
- Optional monthly reminders
- Electronic Prescription Service
- Rapid, reliable delivery
- Discreet packaging
- Convenience orders by subscription



If you would like to find out more about any of our services, please contact our friendly Customer Care Team on **0800 783 1659** or email **info@atos-care.co.uk**

How to order

At Atos Care, we are committed to making life easier for people living with a neck stoma. We offer a variety of options for how you can order your supplies; ranging from a monthly service call to a fully automated monthly order.



A friendly monthly service call

Our helpful Customer Care Team can contact you each month to help you place your order and answer any questions you may have. Monthly calls can help make sure you don't run out of any supplies and give you one less thing to think about each month.



Monthly email or text reminder

Our Customer Care Team can get in touch to provide a list of the items you usually order, help you place the order and answer any questions you may have.



Contact our helpdesk

If you prefer to contact us when you need to order, you can contact us using our convenient freephone phone line, or by text, email or through our website, if you prefer.



Automated convenience order

We can set up a convenient repeat order for you, so that your supplies are automatically ordered and sent to you each month.


We recommend ordering your supplies every **28 days**, as this is the usual GP prescribing cycle. It can take **3-14 days** for us to receive the prescription from your GP, so we offer a range of convenient options to make sure you don't run low on the products you need.

Order process explained

- 1 Place an order with our helpful Customer Care Team by calling us freephone on **0800 783 1659** or emailing **orders@atos-care.co.uk**
- 2 If your order contains new items, we may check suitability with your healthcare professional
- 3 We contact your GP to request the prescription for your items*
- 4 On receiving the prescription**, we send out your supplies directly to your home

Contact us

There are several easy ways to communicate with us - phone, text, email, website or post - depending on what's easiest for you.

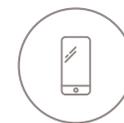

Call or text us

Freephone: 0800 783 1659
Text: 0780 014 0059



Email us

info@atos-care.co.uk



Visit our website

www.atos-care.co.uk



Write to us

Atos Medical, Cartwright House,
Riverside Business Park,
Tottle Road, Nottingham,
NG2 1RT

* This applies if your GP agrees to receive prescriptions from a third part (i.e. us). If this is not the case, you may be required to place the request directly with your GP surgery.

** Our convenient Electronic Prescriptions Service (EPS) means your GP can send the prescription directly through us through a secure system, so that you don't have to collect it and send it on to us.

Laryngectomy




Product shown: Provox Life Home

What is a laryngectomy?

A laryngectomy is an operation which removes the voicebox ('larynx') and is usually carried out because of throat cancer. The larynx is a structure which contains the vocal cords and plays an important role in speaking, breathing and swallowing.

After a laryngectomy, you will breathe through a stoma (opening) in your neck. This means that the air you breathe in and out will not pass through your nose or mouth. You may need to wear some equipment over your stoma to help keep your lungs in good condition and you may notice that your sense of smell and taste are affected. Swallowing (eating and drinking) can be different after a laryngectomy, and unfortunately you will not be able to speak using your usual voice, due to removal of the vocal cords. However, there are a range of options available to help you communicate again after the operation – your healthcare professional will work with you to find the best option for you.

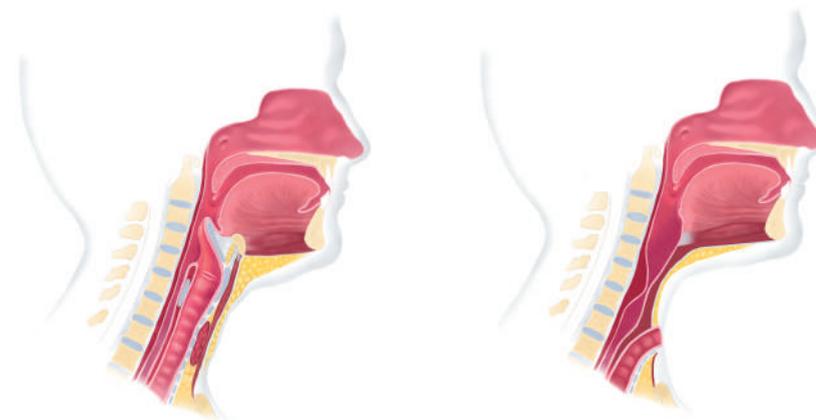

Before total laryngectomy

After total laryngectomy

Looking after your stoma

Looking after your new neck stoma can seem daunting at first, but soon enough it will become second nature.

Before you leave the hospital, your healthcare professionals will show you how to keep your stoma clean. It is recommended to clean your stoma a few times per day, and it needs to be kept free of mucus build-up, which can affect your breathing. The skin around and inside your stoma is very delicate, so it is important to be gentle and careful when undertaking your stoma care.

We suggest using:

A magnifying mirror; a good light or torch; gauze squares, foam swabs or cleaning towels; saline solution or clean water (ask your hospital team for advice). If you are using a baseplate, we recommend using adhesive remover wipes and barrier wipes to help protect your skin.

Remove your baseplate or tube/button:

- If you are wearing a baseplate, apply an adhesive remover wipe over the top of the baseplate and use it to ease the baseplate away from your skin, peeling it off gently. Avoid pulling or ripping off the baseplate, which can damage your skin and cause irritation.
- If you are wearing a lary tube or button, remove it and clean it with a foam swab or tube brush before replacing.



Use an adhesive remover wipe over baseplate




Gently push off the baseplate with remover wipe

Clean your stoma:

- Gently dab around your stoma with a damp gauze square, foam swab or cleaning towel, and gently wipe away any mucus from the skin. Do not pick, scrape or scrub at the skin. If the mucus is dried on, dampen it a little until soft and easy to remove.
- Pat dry the area around the stoma with a dry gauze square or foam swab, or allow it to air dry.
- Use a good light or torch to check that the inside of your stoma is clear. If your stoma has a build up of mucus, you may need to cough it out into a tissue, held over your stoma.



Clean your stoma

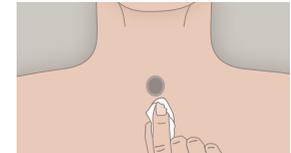

Pat dry your stoma

Replace your baseplate or tube/button:

- If you are going to wear a baseplate, wipe around the stoma with a barrier wipe and allow to dry.
- Pre-warm the baseplate for a few minutes in your hands, peel off the backing and apply to the stoma.
- For best stick, avoid speaking for 10-15 minutes until the baseplate has settled.
- If you are wearing a lary tube or button, apply some lubricating gel to the tube/button and gently insert.



Peel away the liner of the baseplate



Apply the ring of the baseplate as closely as possible to the stoma

What is an HME?


Before a laryngectomy, your nose and mouth help to warm, filter and moisten the air on its way to your lungs. Your lungs work best when they have warm and moist air, as breathing in cool and dry air can lead to problems such as coughing and excess mucus.

After a laryngectomy, a device called a Heat and Moisture Exchanger (HME) can be worn over the stoma to help warm and moisten the air. HMEs contain a special foam, which traps heat and moisture as you breathe out, so that it is transferred to the air that you breathe in. This means that you are breathing in warm and moist air, which helps your lungs to function effectively and can help minimise mucus and coughing.



It is usually recommended to wear HMEs for 24 hours per day, or as close as possible. You may wish to wear a different type of HME depending on what type of activity you are doing. More information about the types of HMEs available can be found on the following pages. An HME needs to be changed at least every 24 hours and is worn with either a sticky adhesive ('baseplate') or a laryngectomy tube or button. The HME clips easily into the baseplate/tube/button and can be removed if you need to cough.

When you first start using an HME you may notice a temporary increase in coughing and mucus. This is quite normal while the lungs adjust and clear out built-up mucus. This should settle down within a couple of weeks and you should start to see the benefits of wearing an HME.

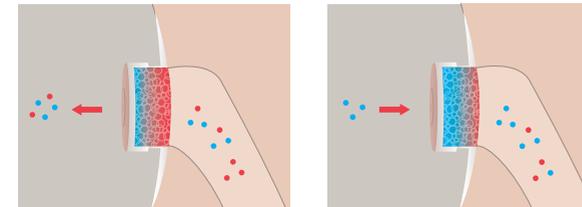

The specially-treated foam inside the HME traps moisture and warmth as you breathe out

The air passes through the foam, warming and moistening the air on its way to the lungs



Product shown: Provox Life Go



Product shown: Provox Life Night

Which HME should I use?


Different HMEs have different benefits, and you may wish to change your HME according to what situation you are in and what activity you are doing. This will enable you to get the best possible benefits from wearing an HME.



Provox® Life™ Home

Home HME is designed to improve your lung health when you are taking it easy. This HME provides high humidification with comfortable breathing throughout the day.

See page p24

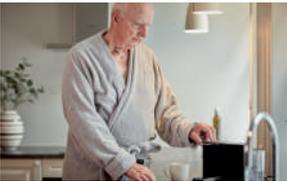

Provox® Life™ Go

When you are out and about, you may prefer to use an HME with more air flow, which is easier to breathe through. These do not offer such effective humidification, but may be more comfortable when you are walking, shopping or going about your day-to-day activities away from home.

See page p24



Provox® Life™ Night

Night HME helps reduce coughing for a more restful night's sleep. This HME offers excellent humidification for improved lung health in a soft, comfortable design.

See page p25



For advice about what products are most suitable for you, please speak to your hospital team.



Provox® Life™ Energy

Energy HME provides low breathing resistance when you are physically active. It has a sporty look and provides good humidification.

See page p25



Provox® Life™ Protect

Protect HME not only heats and humidifies the air you breathe, it also effectively filters bacteria, viruses, dust, and pollen. An HME with comfortable breathing resistance designed to protect against airborne particles.

See page p26



Provox® Life™ FreeHands

FreeHands HME is compatible with FreeHands FlexiVoice, which offers both the freedom to speak hands-free and with finger occlusion.

See page p26



PROVOX[®]
Life Breathe better,
whatever you do

Introducing Provox[®] Life[™] by Atos Medical

Product shown: Provox Life Protect

Provox® Life™ Breathe better, whatever you do

The new Provox Life™ system was developed on the basis of thorough research, and is designed to address the challenges that you experience with HMEs and attachments.


Developed with you in mind, the system offers a complete, flexible solution that allows you to experience a personalised routine, no matter where you are on your recovery journey. There are products for every situation to help you to use HMEs 24 hours a day to build a positive cycle and reduce excessive mucus and coughing.

Please visit our website at www.atos-care.co.uk or call us on **0800 783 1659** for more information on how Provox Life™ can improve your quality of life.



The next generation solution consists of:

- Six high performing HMEs that offer improved humidification and breathability.
- Four newly designed adhesives for a wider range of skin types and stoma contours with improved materials and new SecureFit™ coupling.
- HMEs that are compatible with all adhesives in the Provox Life™ range to offer you a personalised routine.
- Colour coded products and packaging for improved easy identification.

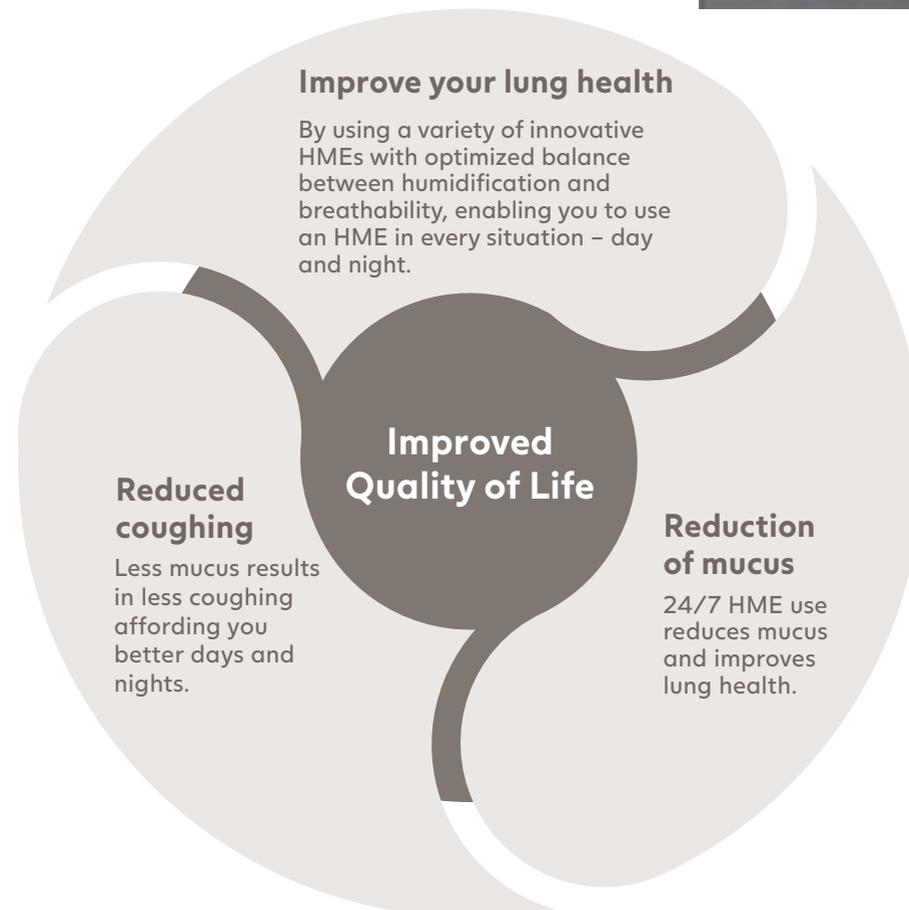

Provox® Life™ - connecting to a better day

Lower profile coupling

The adhesive incorporates a soft, low profile coupling that features a patented SecureFit™ technology for security. It is designed to be compatible with all Provox Life™ HMEs.



Advanced clover-shaped adhesives


A range of innovative clover-shaped adhesives provide a better and more comfortable fit, and are suitable for a wider variety of skin types and stoma contours, so that you can find a personalised solution.



Redesigned dimensions

The redesigned HMEs optimise humidification and improve breathing performance - across all different daily situations. Six high performance HMEs cover your needs from morning to night, whatever you are doing.

PROVOX®
Life Breathe better,
 whatever you do

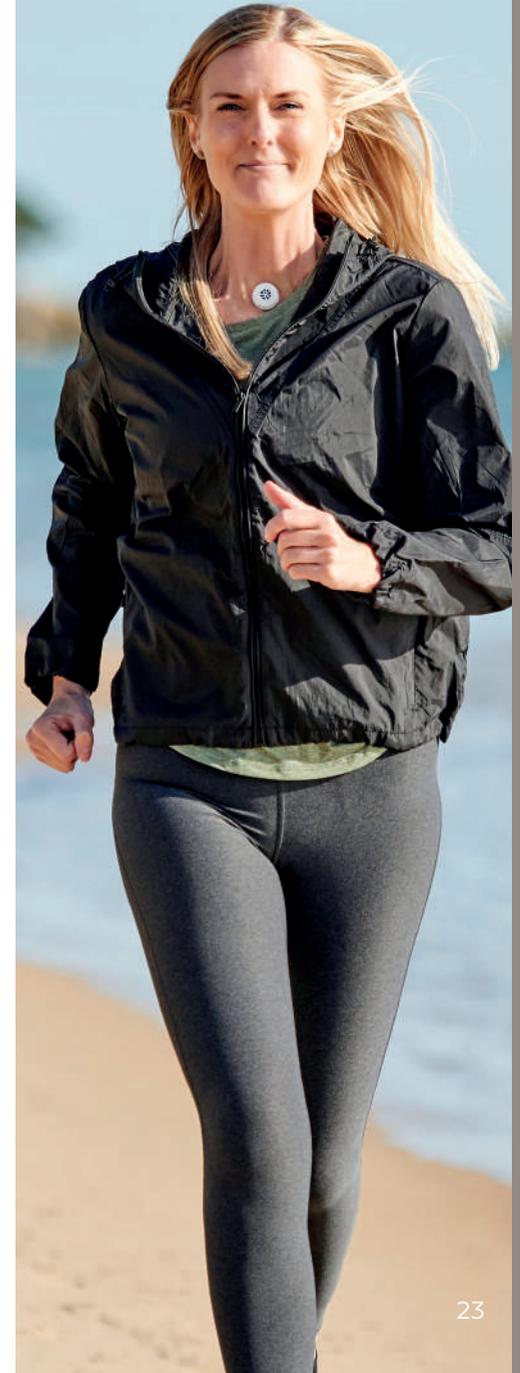

Provox Life™ Heat and Moisture Exchangers

Provox Life™ HMEs mimic the function of your upper airways and cover your needs for every occasion ensuring the right balance between humidification and breathability.



Provox® Life™ Home

This HME is designed to improve your lung health when you are taking it easy at home. Features optimised humidification and breathability for healthier lungs.

- Excellent humidification
- Comfortable breathability
- Low profile and discreet look
- Reduces mucus production and coughing
- Easy to press for speaking with a voice prosthesis

Provox Life Home HME	Pack Size	Order Code
Provox Life Home HME	30 pcs	8311



Provox® Life™ Go

This HME provides lower breathing resistance, helping make it easier to breathe when you are out and about and engaging in daily activities.

- Good breathability
- Good humidification
- Low-profile and discreet look
- Reduces mucus production and coughing
- Easy to press for speaking with a voice prosthesis

Provox Life Go HME	Pack Size	Order Code
Provox Life Go HME	30 pcs	8310

PROVOX
Life

All Provox Life HMEs are now compatible across the whole range of Provox Life adhesives.



Provox® Life™ Night

This HME helps reduce coughing for a more restful nights sleep. It offers high humidification which helps reduce mucus production and coughing, allowing you to rest while improving lung health.

- Excellent humidification
- Comfortable breathability for nighttime breathing
- Reduces mucus production and coughing
- Soft and smooth for nighttime comfort
- Side openings to prevent occlusion while sleeping

Provox Life Night HME	Pack Size	Order Code
Provox Life Night HME	30 pcs	8262



Provox® Life™ Energy

This HME has good humidification and low breathing resistance, which is perfect for when you are physically active.

- High breathability
- Good humidification
- Sporty look
- Reduces mucus production and coughing
- Easy to press for speaking with a voice prosthesis
- Front vents to prevent accidental occlusion during deep breathing

Provox Life Energy HME	Pack Size	Order Code
Provox Life Energy HME	30 pcs	8312

PROVOX
Life

All Provox Life HMEs are now compatible across the whole range of Provox Life adhesives.



Provox® Life™ Protect

This HME effectively filters bacteria, viruses, dust, and pollen. It also has a comfortable breathing resistance that warms and humidifies the air you breathe.

- A highly effective bacterial and viral filter ($\geq 98\%$)*
- Protects you from airborne particles (such as viruses, bacteria, dust and pollen) by effective filtration
- Reduces mucus production and coughing
- Easy to press for speaking with a voice prosthesis

Provox Life Protect HME	Pack Size	Order Code
Provox Life Protect HME	30 pcs	8313



Provox® Life™ FreeHands

Provox FreeHands HME offers both the freedom to speak hands-free and with finger occlusion.

- Reduces mucus production and coughing
- Easy removal of HME without the need for a removal aid

Provox Life FreeHands HME	Pack Size	Order Code
Provox Life FreeHands HME	30 pcs	7440

Provox® FreeHands FlexiVoice™

Gives the ability to speak hands-free. For use with a voice prosthesis.

- Speak hands-free for more natural communication
- Can be covered with a finger if needed
- Quick and easy to set up
- Built in HME for humidification benefits
- Helpful arch accessory helps keep clothing away
- Available in four strengths to suit your needs
- Compatible with Provox Life Adhesives using Provox Life FreeHands HME, or with Provox Adhesives using Provox FreeHands HMEs (see p37)



Provox FreeHands FlexiVoice	Order Code
Provox FreeHands FlexiVoice Light: 1 Light, 1 Arch, 1 Removal Aid	8161
Provox FreeHands FlexiVoice Medium: 1 Medium, 1 Arch, 1 Removal Aid	8162
Provox FreeHands FlexiVoice Strong: 1 Strong, 1 Arch, 1 Removal Aid	8163
Provox FreeHands FlexiVoice XtraStrong: 1 XtraStrong, 1 Arch, 1 Removal Aid	8166

Provox® FreeHands Support™

To help reduce stoma movement, improve voice quality and prevent loss of baseplate seal while speaking hands-free.

- Helps keep a good seal with your baseplate while speaking hands-free
- Adhesive panel fixes to the chest to reduce baseplate movement while speaking
- Three models available to suit different stoma depths



Provox FreeHands Support	Pack Size	Order Code
Provox FreeHands Support Flat	1 pc	8021
Provox FreeHands Support Medium	1 pc	8022
Provox FreeHands Support Deep	1 pc	8023
Provox FreeHands Support Adhesive	15 pcs	8024



*Please note: Since pathogens can enter and leave the human body in other ways (such as the mouth, nose, and eyes), Provox Life Protect HME can never guarantee complete protection.

Provox® Life™ Attachments

The clover-shaped adhesives are designed from high-performance materials to suit different skin types and stoma contours, providing you with a personalised solution. The newly designed adhesives come with improved materials and new SecureFit™ coupling which provides you with comfort and confidence.



Provox® Life™ Standard Adhesive

Standard is an everyday adhesive that provides a strong seal around your stoma. A soft, flexible and low-profile adhesive that forms a strong seal and provides a secure fit.

- Strong seal
- Innovative clover-shaped adhesive for easy application and better fit
- SecureFit technology - comfortable low-profile connector
- Compatible with all Provox Life HMEs
- Ideal for medium to flat stomas



Standard Round



Standard Oval



Standard Plus

Provox Life Standard Adhesive	Pack Size	Order Code
Provox Life Standard Adhesive Round	30 pcs	7460
Provox Life Standard Adhesive Oval	30 pcs	7461
Provox Life Standard Adhesive Plus	30 pcs	7462



Provox® Life™ Sensitive Adhesive

Sensitive is a gentle everyday adhesive for sensitive skin. A comfortable, gentle and low-profile adhesive that forms a good seal for a gentle yet secure fit.

- Good seal
- Hydrocolloid material for sensitive skin
- Innovative clover-shaped adhesive for easy application and better fit
- SecureFit technology - comfortable low-profile connector
- Compatible with all Provox Life HMEs
- Ideal for sensitive skin



Sensitive Round



Sensitive Oval



Sensitive Plus

Provox Life Sensitive Adhesive	Pack Size	Order Code
Provox Life Sensitive Adhesive Round	30 pcs	7463
Provox Life Sensitive Adhesive Oval	30 pcs	7464
Provox Life Sensitive Adhesive Plus	30 pcs	7466



Provox® Life™ Stability Adhesive

A firm everyday adhesive that provides a secure attachment, especially if your stoma is deep. It is ideal for those who speak frequently or speak hands-free.



Stability

- Strong seal
- Innovative clover-shaped adhesive for easy application and better fit
- SecureFit technology - comfortable low-profile connector
- Compatible with all Provox Life HMEs
- Ideal for deep stomas or speaking handsfree
- Reinforced centre panel for extra stability

Provox Life Stability Adhesive	Pack Size	Order Code
Provox Life Stability Adhesive	15 pcs	8263



Provox® Life™ Night Adhesive

Night adhesive allows your skin to recover while you sleep. A skin-friendly hydrogel adhesive that soothes and cools the skin.



Night

- Soft and smooth for nighttime comfort
- Hydrogel material helps soothe and cool the skin while you sleep
- Innovative clover-shaped adhesive for easy application and better fit
- SecureFit technology - comfortable low-profile connector
- Compatible with all Provox Life HMEs
- Ideal for sensitive, irritated skin

Provox Life Night Adhesive	Pack Size	Order Code
Provox Life Night Adhesive	15 pcs	8261



Provox® Life™ LaryTube Standard

For maintaining a stable stoma for those prone to stoma shrinkage.

Available in Standard and Fenestrated (holes for speaking). Secure with Provox Tubeholder or Provox LaryClips (both p33).

'With ring' option allows the tube to be worn with an adhesive baseplate.



LaryTube Standard



LaryTube Standard with Ring

Provox Life LaryTube Standard	ID	OD	L	Order Code
8/27	9.5	12.0	27	7409
8/36	9.5	12.0	36	7410
8/55	9.5	12.0	55	7111
9/27	10.5	13.5	27	7412
9/36	10.5	13.5	36	7413
9/55	10.5	13.5	55	7414
10/27	12.0	15.0	27	7415
10/36	12.0	15.0	36	7416
10/55	12.0	15.0	55	7417
12/27	13.5	17.0	27	7418
12/36	13.5	17.0	36	7419
12/55	13.5	17.0	55	7420

Provox Life LaryTube Standard with Ring	ID	OD	L	Order Code
8/36	9.5	12.0	36	7421
8/55	9.5	12.0	55	7422
9/36	10.5	13.5	36	7423
9/55	10.5	13.5	55	7424
10/36	12.0	15.0	36	7425
10/55	12.0	15.0	55	7426
12/36	13.5	17.0	36	7427
12/55	13.5	17.0	55	7428

ID = inner diameter OD = outer diameter
L = length (in mm)

PROVOX®
Life

PROVOX®
Life



LaryTube fenestrated



LaryTube fenestrated with ring

Provox Life LaryTube Fenestrated	ID	OD	L	Order Code
8/36	9.5	12.0	36	7429
8/55	9.5	12.0	55	7430
9/36	10.5	13.5	36	7431
9/55	10.5	13.5	55	7432
10/36	12.0	15.0	36	7433
10/55	12.0	15.0	55	7434
12/36	13.5	17.0	36	7435
12/55	13.5	17.0	55	7436

ID - inner diameter OD = outer diameter
L = length (in mm)

Provox Life LaryTube Fenestrated with Ring	ID	OD	L	Order Code
8/36	9.5	12.0	36	8048
8/55	9.5	12.0	55	8049
9/36	10.5	13.5	36	8050
9/55	10.5	13.5	55	8051
10/36	12.0	15.0	36	8052
10/55	12.0	15.0	55	8053
12/36	13.5	17.0	36	8054
12/55	13.5	17.0	55	8055

Provox® Life™ LaryButton™

A soft silicone stoma button used to keep the stoma stable in size.



Check with your Healthcare Professional for suitability.

PROVOX
Life

Provox Life LaryButton	OD	L	Order Code
12/8	12	8	8040
14/8	14	8	8041
16/8	16	8	8042
18/8	18	8	8043
12/18	12	18	8044
14/18	14	18	8045
16/18	16	18	8046
18/18	18	18	8047

ID - inner diameter OD = outer diameter
L = length (in mm)



Provox® Life™ Shower

Protects the stoma from water when showering. Compatible with all Provox Life adhesives and tubes.

Provox Life Shower	Pack Size	Order Code
Provox Life Shower	1 pc	8308

Provox® TubeHolder

This can be used to hold a laryngectomy tube or button in place. The discreet clips fit into the connectors of a Provox Life LaryTube and LaryButton.

- Discreet colour
- Side closure with velcro for adjustable length
- High quality material with slight stretch for comfort

Compatible with Provox Life



Provox® Lary Clip

Can be used to hold a laryngectomy tube or button in place. The discreet clips fit into the connectors of a Provox Life LaryTube or LaryButton.

- Discreet colour
- Adhesive backing attaches securely to skin
- Easy fastenings for flexible positioning

Compatible with Provox Life

Provox TubeHolder	Pack Size	Order Code
Provox TubeHolder	1 pc	7668
Provox LaryClip	Pack Size	Order Code
Provox LaryClip	20 pads/8 clips	7669



HMEs, Baseplates and Tubes for Laryngectomy

Product shown: Provox XtraMoist

Provox range by Atos Medical



Provox® XtraMoist HME™

- Excellent humidification
- Reduces coughing and mucus production
- Easy to press for speaking with a voice prosthesis

Provox HME	Pack Size	Order Code
Provox XtraMoist HME	30 pcs	7290

New version available: Provox Life Home (p24).



Provox® XtraFlow HME™

- Low breathing resistance
- Good humidification
- Easy to press for speaking with a voice prosthesis

Provox HME	Pack Size	Order Code
Provox XtraFlow HME	30 pcs	7291

New version available: Provox Life Go (p24).



Provox® Micron HME™

- Highly effective virus and bacteria filter (over 99%)*
- Excellent humidification
- Easy to press for speaking with a voice prosthesis

Provox HME	Pack Size	Order Code
Provox Micron HME	30 pcs	7248

New version available: Provox Life Protect (p26).



Provox® Luna HME™

- Soft and smooth for night-time comfort
- Low breathing resistance for easy night-time breathing
- Side openings to prevent occlusion whilst sleeping

Provox HME	Pack Size	Order Code
Provox Luna HME	30 pcs	8013

New version available: Provox Life Night (p25).



Provox® FreeHands HME® Flow

For humidification when being active.

- To be combined with Provox FreeHands FlexiVoice when speaking hands-free

Provox FreeHands HME	Pack Size	Order Code
Provox FreeHands HME Flow	30 pcs	8221

New version available: Provox Life FreeHands (p26).



Provox® FreeHands HME® Moist

For extra humidification when relaxing.

- To be combined with Provox FreeHands FlexiVoice when speaking hands-free

Provox FreeHands HME	Pack Size	Order Code
Provox FreeHands HME Moist	30 pcs	8220

New version available: Provox Life FreeHands (p26).

*Please note: Since pathogens can enter and leave the human body in other ways (such as the mouth, nose, and eyes), Provox Micron HME can never guarantee complete protection. Please read the instructions for use for guidance.



Check with your Healthcare Professional for suitability.

Provox® FlexiDerm Adhesive™

- Medium stickiness
- Good for flat stomas
- Flexible, translucent material



FlexiDerm Round



FlexiDerm Oval



FlexiDerm Plus

Provox Adhesive FlexiDerm	Pack Size	Order Code
Provox FlexiDerm Round	20	7253
Provox FlexiDerm Oval	20	7254
Provox FlexiDerm Plus	20	7331

New version available: Provox Life Standard (p28).

Provox® OptiDerm Adhesive™

- Low stickiness
- Suitable for use after surgery or radiotherapy
- Hydrocolloid material, gentle on sensitive skin



OptiDerm Round



OptiDerm Oval



OptiDerm Plus

Provox Adhesive OptiDerm	Pack Size	Order Code
Provox OptiDerm Round	20	7255
Provox OptiDerm Oval	20	7256
Provox OptiDerm Plus	20	7332

New version available: Provox Life Sensitive (p29).

Provox® StabiliBase Adhesive™

- For speaking hands-free
- Curved, reinforced centre panel
- High stickiness
- Ideal for deep stomas

New version available: Provox Life Stability (p30).



Provox® StabiliBase™ OptiDerm Adhesive™

- Low stickiness
- Curved, reinforced centre panel
- Ideal for sensitive skin and deep stomas



Provox® XtraBase Adhesive™

- High stickiness
- Round, reinforced centre panel
- Ideal for medium-deep stomas or speaking hands-free



Provox® Luna Adhesive™

- Hydrogel material soothes skin and helps to reduce skin irritation.
- Soft and smooth for nighttime comfort
- Innovative clover-shaped adhesive for easy application and better fit

New version available: Provox Life Night (p30).



Provox Adhesive	Pack Size	Order Code
Provox StabiliBase	15 pcs	7289
Provox StabiliBase OptiDerm	15 pcs	7318
Provox XtraBase	20 pcs	7265
Provox Luna Adhesive	15 pcs	8014
Provox Adhesive Strips	10 pcs	8015

Provox® LaryTube™ Standard

A soft silicone tube used to keep the stoma stable in size. Suitable for use with or without a voice prosthesis.



Provox® LaryTube™ Standard with Ring

A soft silicone tube used to keep the stoma stable in size. Suitable for use with or without a voice prosthesis. Attaches directly into Provox adhesive baseplates; no need for neckbands.



Provox® LaryTube™ Fenestrated

A soft silicone tube used to keep the stoma stable in size. Small holes (fenestrations) are designed for use with a voice prosthesis.



Provox® LaryButton™

A soft silicone stoma button used to keep the stoma stable in size.



LaryTube Standard	ID	OD	L	Order Code
8/27	9.5	12.0	27	7601
8/36	9.5	12.0	36	7602
8/55	9.5	12.0	55	7603
9/27	10.5	13.5	27	7605
9/36	10.5	13.5	36	7606
9/55	10.5	13.5	55	7607
10/27	12.0	15.0	27	7609
10/36	12.0	15.0	36	7610
10/55	12.0	15.0	55	7611
12/27	13.5	17.0	27	7613
12/36	13.5	17.0	36	7614
12/55	13.5	17.0	55	7615

ID = inner diameter OD = outer diameter
L = length (in mm)

LaryTube Standard With Ring	ID	OD	L	Order Code
8/36	9.5	12.0	36	7624
8/55	9.5	12.0	55	7625
9/36	10.5	13.5	36	7626
9/55	10.5	13.5	55	7627
10/36	12.0	15.0	36	7628
10/55	12.0	15.0	55	7629
12/36	13.5	17.0	36	7630
12/55	13.5	17.0	55	7631

LaryTube Fenestrated	ID	OD	L	Order Code
8/36	9.5	12.0	36	7637
8/55	9.5	12.0	55	7638
9/36	10.5	13.5	36	7640
9/55	10.5	13.5	55	7641
10/36	12.0	15.0	36	7643
10/55	12.0	15.0	55	7644
12/36	13.5	17.0	36	7646
12/55	13.5	17.0	55	7647

ID = inner diameter OD = outer diameter
L = length (in mm)

LaryButton	OD	L	Order Code
12/8	12	8	7671
14/8	14	8	7672
16/8	16	8	7673
18/8	18	8	7674
12/18	12	18	7685
14/18	14	18	7686
16/18	16	18	7687
18/18	18	18	7688



Check with your Healthcare Professional for suitability.

SpiroTect HME




Push2Speak

A HME which is pressed for speaking with a voice prosthesis.

SpiroTect HMEs	Pack Size	Order Code
Push2Speak HME	30 pcs	PTC1005

SpiroTect Baseplates

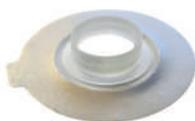

Extra Sticky Baseplate

Foam baseplate with extra adhesive.



Convex Baseplate

Curved centre panel suitable for deep stomas.



Fabric Baseplate

Soft fabric material for comfort.



Sensitive Baseplate

Hydrocolloid baseplate, designed for sensitive skin.



Clear & Strong Baseplate

Transparent with strong adhesive.

SpiroTect Baseplates	Oval	Round	Convex	Pack Size
Extra Sticky Baseplate		PTW501		20 pcs
Convex Baseplate			PTW700	20 pcs
Fabric Baseplate	PTW100	PTW101		20 pcs
Sensitive Baseplate	PTW200	PTW201		20 pcs
Clear and Strong Baseplate	PTW300	PTW301		20 pcs

Blom-Singer HME



HumidiFilter Holder
Reusable holder that houses the foam filter. Designed for daytime use.



HumidiFilter Holder Foam Filters
A foam disk that can be worn in the re-usable HumidFilter Holder.



HME ClassicFlow Cartridge
Can be pressed down for speaking with a voice prosthesis.



Hands-Free ATSVII Valve Kit

For handsfree speech. Check with professional for suitability.



Hands-Free ATSVII HumidiFilter Foam Filters

Replacement foam disks for use with the ATSVII Cap.



Hands-Free ATSVII Replacement Faceplate

Replacement faceplate for the ATSVII Handsfree valve.



ATSVII Cap & 7 Foam Filters

Twist-on plastic cap with foam disks, for use with the ATSVII Handsfree valve.

Blom-Singer HMEs	Pack Size	Order Code
HumidiFilter Holder	1 pc	BE1060
HumidiFilter Holder Foam Filters	30 pcs	BE1080
HME ClassicFlow Cartridge	30 pcs	BE1055EZ
Hands-Free ATSVII Valve Kit	1 pc	BE8025H
Hands-Free ATSVII HumidiFilter Foam Filters	30 pcs	BE1030
Hands-Free ATSVII Replacement Faceplate	1 pc	BE8026H
ATSVII Cap & 7 Foam Filters	1 pc	BE1010

Blom-Singer Baseplates



Round Oval Oval extra Round Oval Oval extra

AccuFit Adhesive Housing
Flexible, translucent adhesive.

HydroFit Hydrocolloid
Adhesive Housing
Hydrocolloid adhesive suitable for
sensitive skin.



Tape Adhesive
Discs
Double-sided
adhesive tape disks
for use with the
Tracheostoma Valve
Housing.

Tracheostoma
Valve Housing
A reusable silicone
housing for use with
Tape Adhesive Discs.
Fits with the ATSVII
or Humidfilter Holder.

Kapi-Gel
Tracheostoma
Spacer
Helps to fill in the area
around deeper stomas
for a better seal with
any type of baseplate.

Blom-Singer Adhesives	Large	Standard	Oval extra	Oval	Round	Pack Size
AccuFit Adhesive Housing			BE6084	BE6083	BE6082	30 pcs
HydroFit Hydrocolloid Adhesive Housing			BE6087	BE6086	BE6085	30 pcs
Tracheostoma Valve Housing	BE6039	BE6039				30 pcs
Tape Adhesive Discs Regular	BE6042	BE6042				30 pcs
Tape Adhesive Discs Thin	BE6044	BE6044				30 pcs
Tape Adhesive Discs Thin Heavy Duty	BE6035	BE6034				30 pcs
KapiGel			Internal opening diameter: 8mm	Internal opening diameter: 12 mm		Pack Size
Kapi-Gel Thick			LATNG4008	LATNG4012		10 pcs
Kapi-Gel Thin			LATNG2008	LATNG2012		10 pcs

Laryngectomy Accessories

Tube Swabs

For cleaning all types of PVC and silicone laryngectomy tubes.



Tube Swabs	For Tube Size	Pack Size	Order Code
Provox Swabs Small	4 - 5mm	50 pcs	8250
Provox Swabs Medium	6 - 8mm	50 pcs	8251
Provox Swabs Large	9 - 10mm	50 pcs	8252
Provox Swab Extra Large	11 - Above	50 pcs	8258
Trachi-Swab Small	4 - 5mm	50 pcs	TRSWB0150
Trachi-Swab Large	5 - 7mm	50 pcs	TRSWB0250
Trachi-Swab Extra Large	8 - Above	50 pcs	TRSWB3050
Insight Tube Swab Small	Up to 6mm	10 pcs	TCS/950
Insight Tube Swab Large	6 - 9mm	10 pcs	TCS/900
SpiroTect Tube Swab		50 pcs	PTA300

Tube Brushes

These brushes are used to clean laryngectomy tubes to help keep them clean and free of mucus or debris.

Tube Brush	Size	Pack Size	Order Code
Provox TubeBrush	8mm	6 pcs	7660
Provox TubeBrush	12mm	6 pcs	7661
Kapitex Tracheo	8mm	2 pcs	TRACC0001
Kapitex Tracheo	10mm	2 pcs	TRACC0002
Kapitex Tracheo	12mm	2 pcs	TRACC0003
Kapitex Tracheo	14mm	2 pcs	TRACC0004



Voice Prosthesis Brushes

Designed to help clean your voice prosthesis. It is recommended to use a brush that matches the size and brand of your voice prosthesis.

Voice Prosthesis Brush	Voice prosthesis length	Pack Size	Order Code
Provox Brush	4.5 - 10mm	6 pcs	7204
Provox Brush XL	12 - 18mm	6 pcs	7225
Blom-Singer Brush	4 - 8mm	3 pcs	IN4064
Blom-Singer Brush	10 - 16mm	3 pcs	IN4065
Blom-Singer Brush	18 - Above	3 pcs	IN4066
Voiceline Brush		10 pcs	LAPCB0003



Provox® Blom-Singer Voiceline

Shower Protectors

Protects the stoma from water when showering.



Provox® Life
For use with the Provox Life range.



Provox®
For use with the Provox range.



Provox® Luna
For use with Provox Luna baseplate and Adhesive Strips.



SpiroTect
For use with SpiroTect baseplates.



Cascade
A flexible, waterproof cover which can be worn over the stoma.

Shower Protectors	Pack Size	Order Code
Provox Life Shower	1 pc	8308
Provox ShowerAid	1 pc	7260
Provox Luna ShowerAid	1 pc	8016
SpiroTect Shower Protector	2 pcs	PTA100
Cascade Shower Protector	2 pcs	AS3835



Check with your Healthcare Professional for suitability.

Stoma Protectors

When worn tightly and completely sealed around the stoma, stoma protectors and covers can help to warm, moisten and filter inhaled air. The Provox Protectors are hand washable up to 3 times. They can also be worn over a HME for additional protection when out during cold weather.

Provox® Protector

- Thick, absorbent foam stoma protector
- Ideal for those with lots of secretions
- Comfortable and discreet
- High quality material with soft-lock mechanism




Provox® Protector Slim

- Lightweight, fabric stoma protector
- Ideal for those with minimal secretions
- Comfortable and discreet
- High quality material with soft-lock mechanism
- Easy to breathe through



Provox® Protector Air

- Light, foam stoma protector
- Ideal for those with thick or sticky secretions - easily wipe-able
- Comfortable and discreet
- High quality material with soft-lock mechanism
- Easy to breathe through

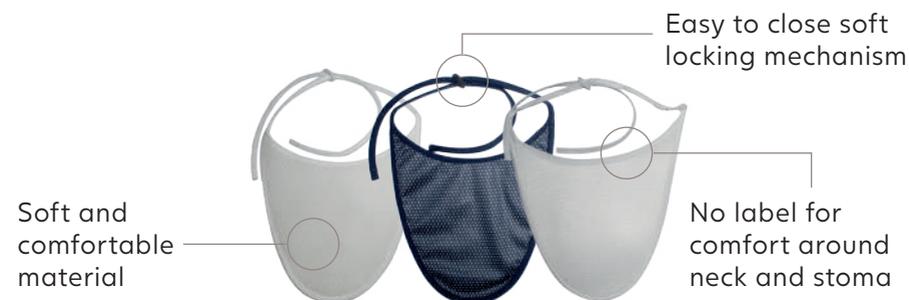

Provox Protectors	White	Blue	Pack Size
Provox Protector Small	7385		10 pcs
Provox Protector Large	7386		10 pcs
Provox Protector Slim Small	7387	7388	10 pcs
Provox Protector Slim Large	7389	7390	10 pcs
Provox Protector Air Small	7391	7392	10 pcs
Provox Protector Air Large	7393	7394	10 pcs



Buchanan Lite
Foam stoma protector



SpiroTect
Foam stoma protector




Hirst
Two layers of knitted net material, no foam, Velcro fastening

Stoma Covers	Colour	Small	Medium	Large	Pack Size	
SpiroTect	Blue	PTA700		PTA702	10 pcs	
Buchanan Lite	White	LABLT2001		LABLT1001	10 pcs	
Buchanan Lite	Beige	LABLT2003		LABLT1003	10 pcs	
Buchanan Lite	Blue	LABLT2002		LABLT1002	10 pcs	
Buchanan	White	BUPO002		BUPO001	10 pcs	
Buchanan Deltanex	White	DNX0001			10 pcs	
Stoma Covers	Beige	White	Black	Blue	Grey	Pack Size
Hirst		BIB				10 pcs

Neckbands


Freevent® Neckband one-piece with twill ties

High quality adjustable neckband with twill ties suitable for use with silicone tubes. Simply cut and adjust to the exact size needed, available in both adult and paediatric sizes.

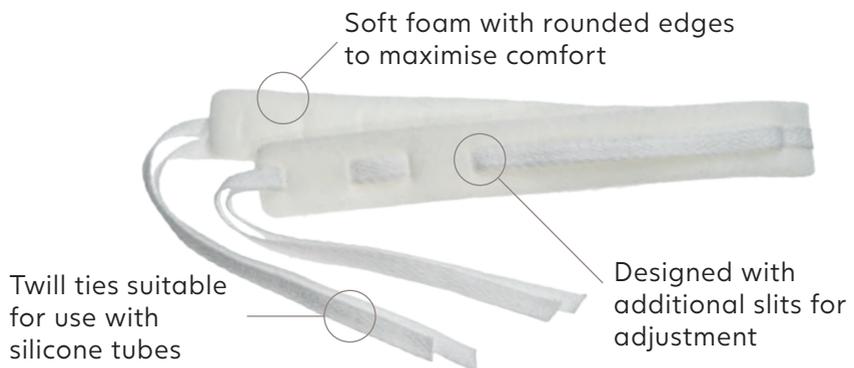

Freevent® Neckband two-piece with VELCRO® brand fastening

High quality adjustable neckband for secure tube positioning. Simple application with VELCRO® fastening, adjustable to adult or paediatric neck size.

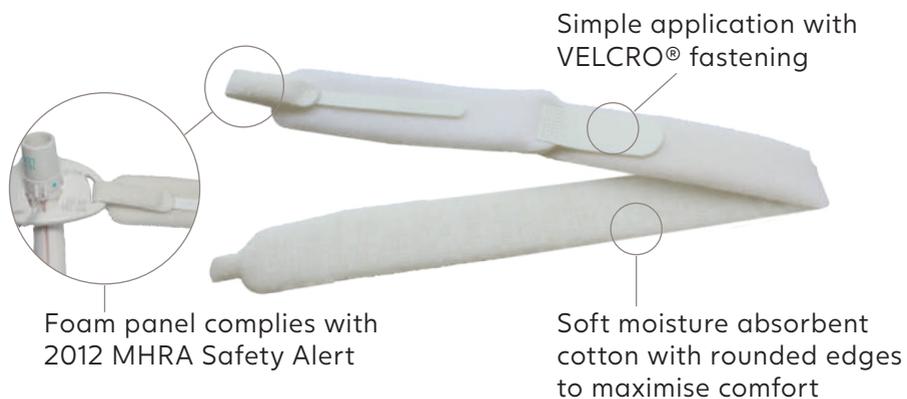

Freevent Neckbands	Adult	Paediatric	Pack Size
Freevent one-piece Freevent one-piece with twill ties	1661	1651	10 pcs
Freevent two-piece with velcro fastening	1662	1652	10 pcs

Neckbands

To help secure a laryngectomy tube in position.



Provox® TubeHolder
For use with Provox Life and Provox tubes or buttons.



Insight



SpiroTect Two-piece



Trachi-Hold



Marpac One-Piece



Marpac Two-Piece

Tube Holders	Adult	Paediatric	Neo-natal	Pack Size
Provox TubeHolder	7668			1 pc
Insight	TH100	TH150		10 pcs
SpiroTect Two-Piece	PTA201	PTA301		10 pcs
Trachi-Hold	TRACC0012	TRACC0014		10 pcs
Marpac One-Piece		107	106	25 pcs
Marpac Two-Piece	201	203MARPAC		10 pcs

Foam Covers

Small foam squares with self-adhesive strip(s), may be worn over the stoma.



Provox®
Tracheofix
2 Strips

Provox®
Tracheofix
1 Strip

Laryngofoam

Purifoam

Foam Protectors	Beige	Ivory	Pack Size
Provox Tracheofix 55mm x 60mm (1 Strip)	1429H		10 pcs
Provox Tracheofix 70mm x 70mm (1 Strip)	1430H		10 pcs
Provox Tracheofix 55mm x 60mm (2 Strips)	1432H		10 pcs
Provox Tracheofix 70mm x 70mm (2 Strips)	1433H		10 pcs
Laryngofoam 51mm x 62mm (1 Strip)	LALFF0003	LALFF0002	30 pcs
Laryngofoam 63mm x 65mm (1 Strip)	LALFF0023		30 pcs
Purifoam 65mm x 6.5mm (1 Strip)		CR3853	30 pcs

Please note: Foam protectors do not provide the same humidification benefits as a HME (see p20-22).

Lubricants

Lubricating water soluble jelly to help reduce friction when inserting tubes and buttons. All are sterile until opened.



OptiLube	Size	Pack Size	Order Code
OptiLube Sachet	5g	150	1120
OptiLube Tube	42g	1 pc	1121
OptiLube Tube	82g	1 pc	1122

Skin Wipes and Creams

Skin wipes and creams to help maintain a healthy skin care routine. For more information see p16-17.



Provox® Skin Barrier

- Leaves a protective film on the skin to help prevent skin irritation
- Use prior to applying adhesive baseplate
- Convenient portable sachets
- No sting formula, safe for use around the stoma



Provox® Adhesive Remover

- Helps to remove adhesive baseplates comfortably and safely
- Reduces risk of skin irritation
- Convenient portable sachets
- No sting formula, safe for use around the stoma



Provox® Cleaning Towel

- A convenient and hygienic way to clean around the stoma
- Convenient, portable pack size, ideal for handbags, care bags and pockets
- No sting formula
- Does not leave any residue on the skin; ideal for getting a good baseplate seal



SpiroTect Barrier Wipe



SpiroTect Adhesive Remover



Cavilon Barrier Cream



Cavilon Barrier Film Pre-soaked swabs

Skin Wipes and Creams	Pack Size	Order Code
Provox Skin Barrier	50 pcs	8011
Provox Adhesive Remover	50 pcs	8012
Provox Cleaning Towel	10 pcs	7244
SpiroTect Adhesive Remover	30 pcs	PTA600
SpiroTect Barrier Wipe	30 pcs	PTA500
Cavilon Barrier Cream	92 g	3392G
Cavilon Barrier Film	1 ml	3343P

Silicone Adhesives and Tapes

Silicon adhesive can be used with baseplates to help achieve a very secure seal; particularly helpful if speaking hands-free.



Provox® silicon glue



Insil silicon tape

Silicone Adhesives	Size	Pack Size	Order Code
Provox glue	30 ml	1 pc	7720
Insil silicon tape	2 cm x 3 m	1 pc	Insil200
Insil silicon tape	4 cm x 1.5 m	1 pc	Insil400



Check with your Healthcare Professional for suitability.

Irrigation Fluid

Saline (salty water) sachets and pods can be used to wash wounds and are suitable to clean around a tracheostoma.



Irripod



Normasol

Irrigation Fluid	Size	Pack Size	Order Code
Irripod	20 ml	25 pcs	IRR24N
Normasol	25 ml	25 pcs	NOR206B

Gauze Swabs



Spec 28 Non Woven Swabs

Non-woven gauze swabs (squares).

- Gauze swabs have a number of uses for cleaning and caring for the stoma and associated equipment.
- Can be used in conjunction with foam swabs
- Highly absorbent and low-linting

Gauze Swab	Size	Pack Size	Order Code
Gauze Swab	100 mm x 100 mm	100 pcs	SWA264T

Sterile Dressing Pack Spec 10

Pre-packed, sterile dressing pack.



Contains:

- Absorbent paper towel (500mm x 450mm) x1
- Gauze swabs (100mm x 100mm; 12ply) x4
- Cotton balls x4, absorbent dressing pad (100 x 200mm) x1
- Sterile field liner

Sterile Dressing Pack Spec 35

Pre-packed, sterile dressing pack.



Contains:

- Absorbent paper towel (500mm x 450mm) x1
- Gauze swabs (100mm x 100mm; 4ply) x4
- Cotton balls x4
- Absorbent dressing pad (100 x 200mm) x1
- Sterile field liner

Dressing Packs	Pack Size	Order Code
Sterile Dressing Pack Spec 10	12 pcs	DP SPEC 10
Sterile Dressing Pack Spec 35	12 pcs	DP SPEC 35



Support and Services

Product shown: Provox Life Go


Our Customer Care Team

Our helpful and friendly Customer Care Team are here to support you and can help you with understanding, ordering and getting the best out of your products.

They can answer any questions you may have about the products in our range, help you decide what quantities to order, and can liaise with your hospital team to understand what products are most suitable for you.

Each member of our Customer Care Team receives in-depth training to help them understand life with a laryngectomy or tracheostomy and to give them a detailed understanding of the products we supply. They also receive regular call coaching and training to help maintain a high standard of customer service.

The Customer Care Team is led by Lizz Summers, a Speech and Language Therapist with many years' of experience helping people who have had a laryngectomy or tracheostomy.

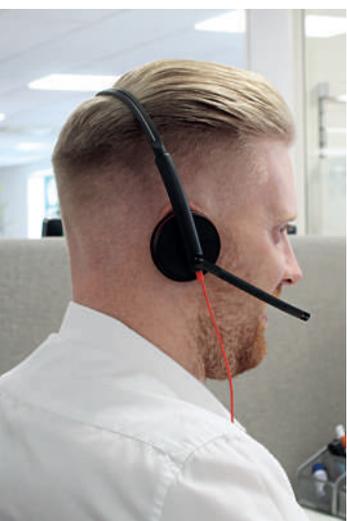

You will have a dedicated Customer Care Representative, who will work with you each month to order the supplies that you need and offer ongoing support.

Customer Care Team Leader, Luke Chadbourne

“ Everything is excellent. My Customer Care Rep takes care of all of my needs. Thanks to them, I am coping very well.

Anonymous feedback



Customer Care Representative, Sophie Coates

Meet our Atos Care Nurses

At Atos Care, we have a team of in-house Nurses to help support you with adjusting to life after your laryngectomy or tracheostomy.

Our experienced and knowledgeable Nurses can help with a range of important aspects such as:

- Learning to look after your stoma or tracheostomy tube
- Getting started with HMEs
- Keeping the skin around your stoma or tracheostomy tube healthy
- Communicating after a laryngectomy or tracheostomy

Our Nurses can visit you at home or support you via phone call, text message, email or video call.

Please note: At present, our Nursing service is not available in all areas due to local regulations. Please contact us to see if the Nursing service is available in your region.



We have recently received accreditation from the Care Quality Commission; an independent regulator of Health and Social Care in England and Wales which ensures that services are safe, effective, compassionate and high quality.



A visit from the specialist Nurse helped me greatly, gave me confidence.

Tracheostomy patient



Atos Care Nurse, Rachel McGowan

MyLife App

Hints and tips on the go

Did you know that we have a helpful App, full of informative videos, tutorials and tips about living with a laryngectomy?

Featuring real patient stories as well as presentations from our in-house Speech Therapists and guest experts, our interactive App is great for hints and tips on the go.


The App is divided into three main sections making it easy for you to find what you are looking for:



- My Learning
 - Articles and original videos with patients and clinicians sharing advice and unique stories. Topics include breathing, speaking, care and daily life
- My Goals
 - A tool to help you get used to HMEs and to help you reduce your coughing. The tool offers useful tips and daily tracking of your progress
- Products
 - A presentation of the Provox® Life products with information to help understand how to use them best within your daily life

Please note: At present, our App mainly contains content relevant for people with a laryngectomy and may be less relevant for those with a tracheostomy.

To download the MyLife App, simply head to the App Store or Google Play store on your smartphone and search for **'Atos MyLife'**

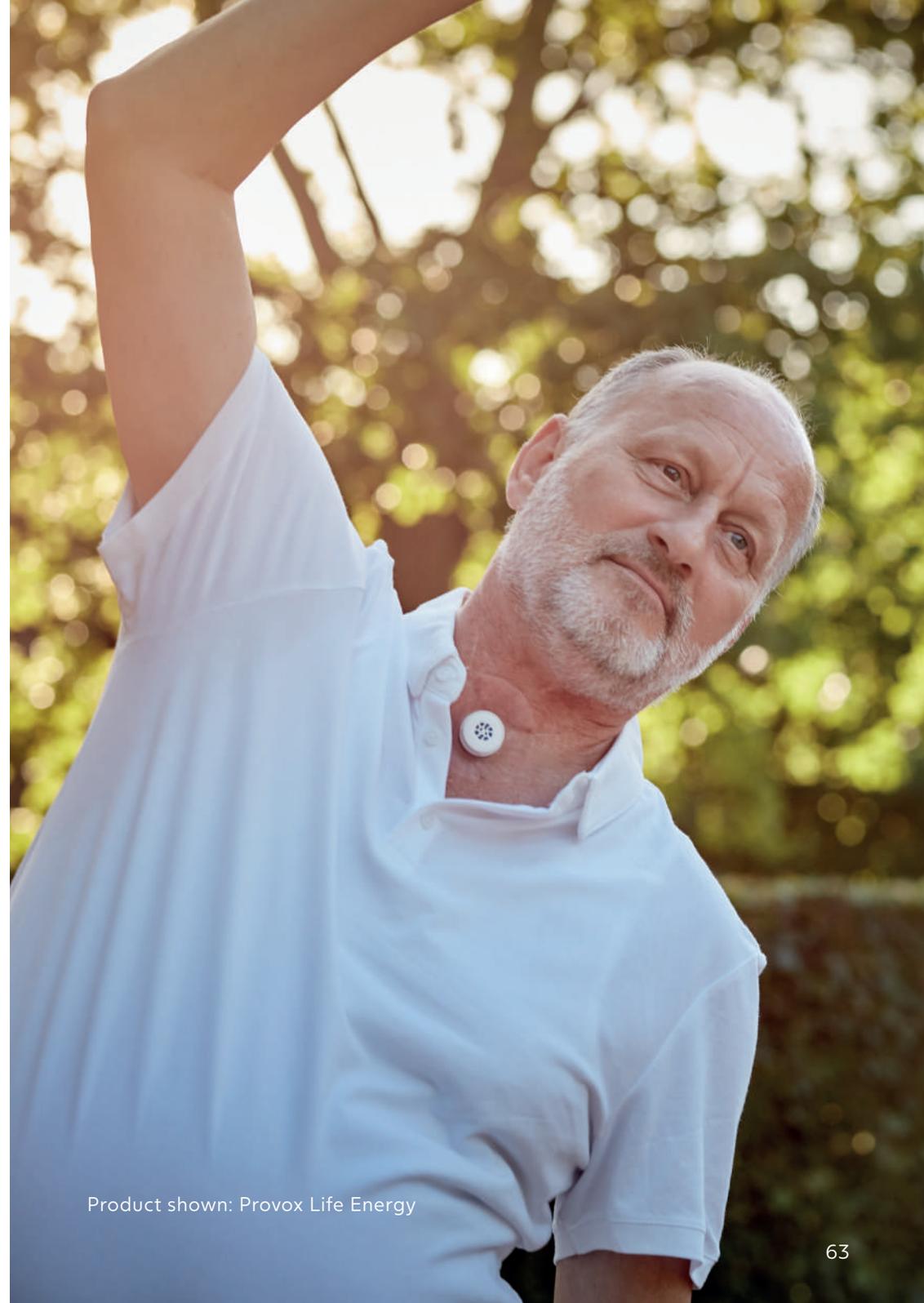

Product shown: Provox Life Energy

Events

Connecting you to others


We run a range of events, both online and in person, to help connect you to others who know and understand about life after a laryngectomy or tracheostomy.

These events include helpful presentations about relevant topics, as well as a chance to ask questions and hear from other people who live with a neck stoma and healthcare professionals.

Our topics are decided by the people who attend our events and previous topics have included:

- Looking after your lungs
- Travel
- Breathing exercises
- Dealing with the unexpected

You can sign up to an event or find out more by contacting us on **0800 783 1659** or emailing **events.uk@atosmedical.com**

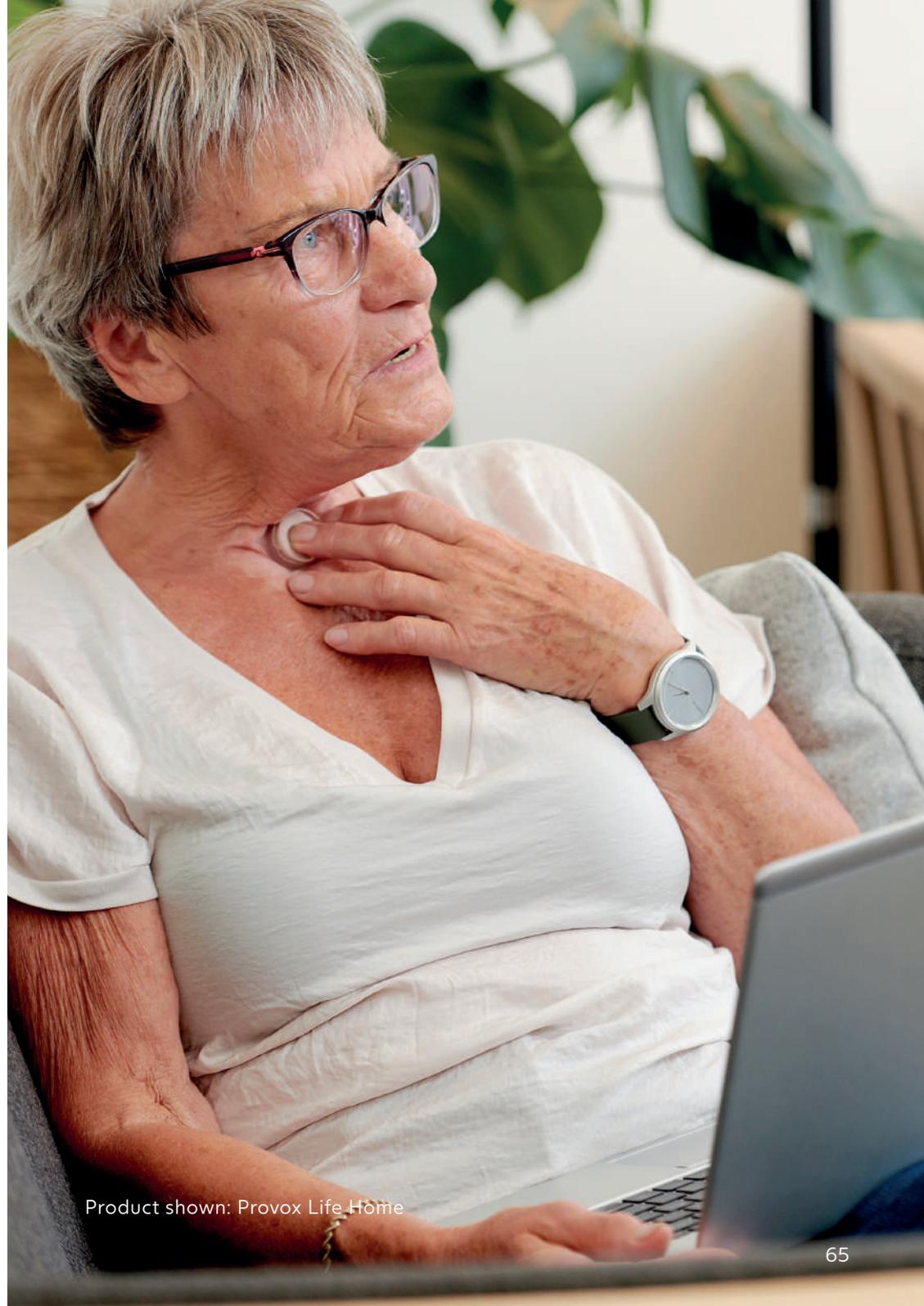

Product shown: Provox Life Home

Tracheostomy




What is a tracheostomy?

A tracheostomy is an operation to make an opening and insert a tube into the neck, usually carried out to help with breathing. After a tracheostomy, you will breathe through the tracheostomy tube. You may need to wear some equipment over your tracheostomy to help keep your lungs in good condition and you may notice a change or reduction in your sense of smell and taste because there may be less air passing through your nose and mouth.




Product shown: Freevent DualCare

There are lots of different types of tracheostomy tube and lots of reasons why people have a tracheostomy, so each patient is different. Some people may find eating and drinking (swallowing) difficult, or even impossible, after a tracheostomy. Some people may be able to speak after a tracheostomy using equipment and others may not; however, there are a range of options available to help you communicate again after the operation – your healthcare professional will work with you to find the best option for you.

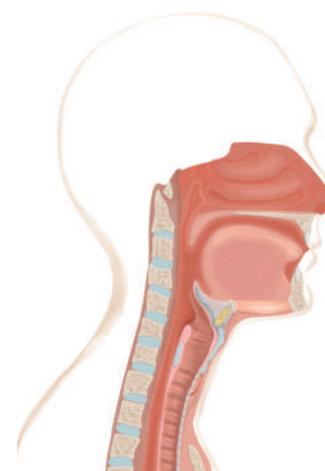

Before a tracheostomy

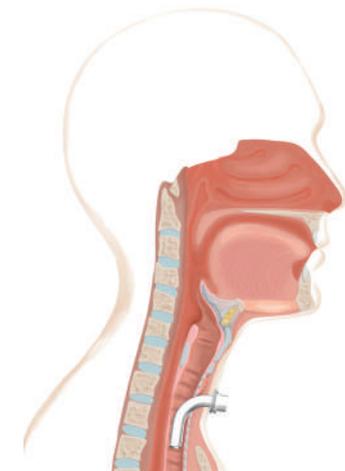

After a tracheostomy

Product shown: Freevent DualCare

Breathing after a tracheostomy

Before a tracheostomy, your nose and mouth help to warm, filter and moisten the air on its way to your lungs. Your lungs work best when they have warm and moist air.


Breathing in cool and dry air can lead to problems such as coughing and excess mucus. After a tracheostomy, a device called a Heat and Moisture Exchanger (HME) can be worn over the tracheostomy tube* to help warm and moisten the air. HMEs contain a special foam, which traps heat and moisture as you breathe out, so that it is transferred to the air that you breathe in. This means that you are breathing in warm and moist air, which helps your lungs to function effectively and can help minimise mucus and coughing.




Freevent XtraCare with O2 adapter

It is usually recommended to wear HMEs for 24 hours per day, or as close as possible. You may wish to wear a different type of HME depending on what type of activity you are doing. More information about the types of HMEs available can be found on p62-64. An HME needs to be changed at least every 24 hours and can be removed if you need to cough, or to nebulise.

Many people with a tracheostomy also use oxygen or a ventilator to help with breathing. Your hospital team can help you establish a routine based on your breathing needs.

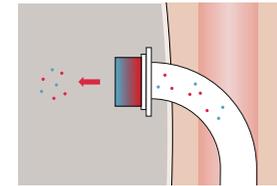


The specially-treated foam inside the HME traps moisture and warmth as you breathe out

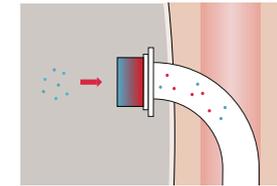

The air passes through the foam, warming and moistening the air on its way to the lungs



Product shown: Freevent XtraCare

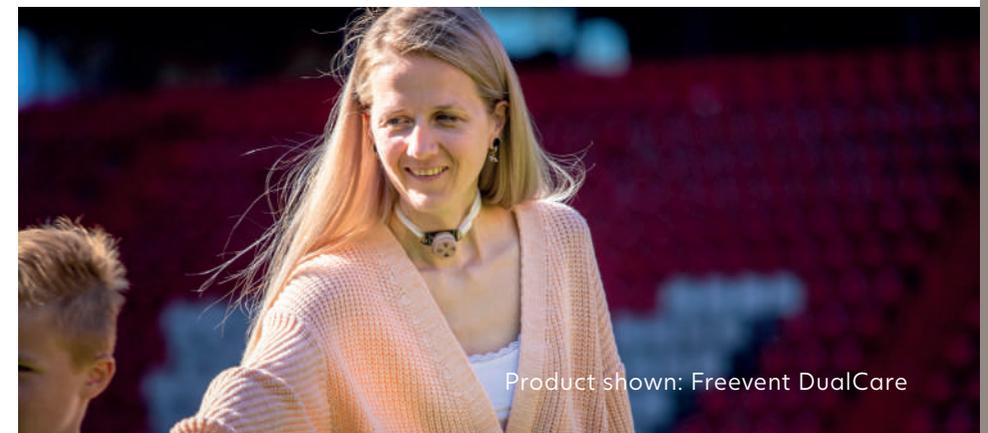

Product shown: Freevent DualCare

* Some tracheostomy tubes are not compatible with HMEs or require an adapter. Please speak to your Healthcare Professional or our Customer Care Team for more information.



HMEs for Tracheostomy

Product shown: Freevent DualCare

Freevent range by Atos Medical

HMEs help to warm and moisten the air you breathe. Research shows that wearing an HME 24 hours per day helps to reduce coughing and mucus.

Freevent® XtraCare HME™

This HME contains a highly effective filter which stops over 99% of viruses, bacteria, pollen and other particles from entering the airway.

It is suitable for non-ventilated tracheostomy patients and can be used when you require extra protection.

- Highly effective virus and bacteria filter (over 99%)*
- Excellent humidification
- Suitable for adults and paediatrics
- White option for discreet appearance
- Blue option is easily seen
- Can be used with oxygen using the add-on oxygen (O₂) adapter
- 15mm connector fits most tracheostomy tubes



ProTrach	Pack Size	Order Code
Freevent XtraCare HME White	30 pcs	7767
Freevent XtraCare HME Blue	30 pcs	7768
Freevent O ₂ Adapter	10 pcs	7769

*Please note: Since pathogens can enter and leave the human body in other ways (such as the mouth, nose, and eyes), Provox Freevent HME can never guarantee complete protection. Please read the instructions for use for guidance.

Freevent® DualCare™ Speaking valve and HME

Freevent DualCare combines a speaking valve and HME, enabling you to speak and humidify your lungs with the same device. Suitable for both adult and paediatric use for non-ventilated tracheostomy patients.

Speaking mode

Speaking mode enables you to speak hands-free with a simple twist of the lid.

Breathing mode

Breathing mode enables you to use the HME function to warm and humidify the air when you are not speaking.

- Enables hands-free speech
- Can be worn at night using the Freevent HME DigiTop
- Tinted option for discreet appearance
- Blue option is easily seen
- 15mm connector fits most tracheostomy tubes



ProTrach DualCare HME	Pack Size	Order Code
Freevent DualCare HME	30 pcs	7742
Freevent DualCare Tinted: 1 Speaking Valve, 1 HME DigiTop Tinted, 1 Connection Strap	1 pc	7744
Freevent DualCare Blue: 1 Speaking Valve, 1 HME DigiTop Blue, 1 Connection Strap	1 pc	7755



Check with your Healthcare Professional for suitability.

Please note: The DualCare is not suitable for patients with an inflated cuffed tracheostomy tube and suitability must be confirmed by a healthcare professional. The HME function is not in operation when the DualCare is in speaking mode.

TrachPhone HME™

A very popular, effective and lightweight HME for tracheostomy patients. Featuring a finger operated speaking valve, cough valve, suction port and oxygen port.

- Lid can be easily pressed for speaking
- Convenient suction port; no need to remove HME for suctioning
- Excellent humidification
- Integrated oxygen connector
- 15mm connector fits most tracheostomy tubes



TrachPhone	Pack Size	Order Code
TrachPhone HME	30 pcs	7707

Shiley Tracheostomy Tubes

Shiley tracheostomy tubes come in a range of types and sizes. To help ensure you get the right fit for safety and comfort, the code for your Shiley tube must be provided by your Healthcare Professional.




Tracheostomy Accessories



Check with your Healthcare Professional for suitability.

Tube Swabs

For cleaning all types of PVC and silicone tracheostomy inner tubes.

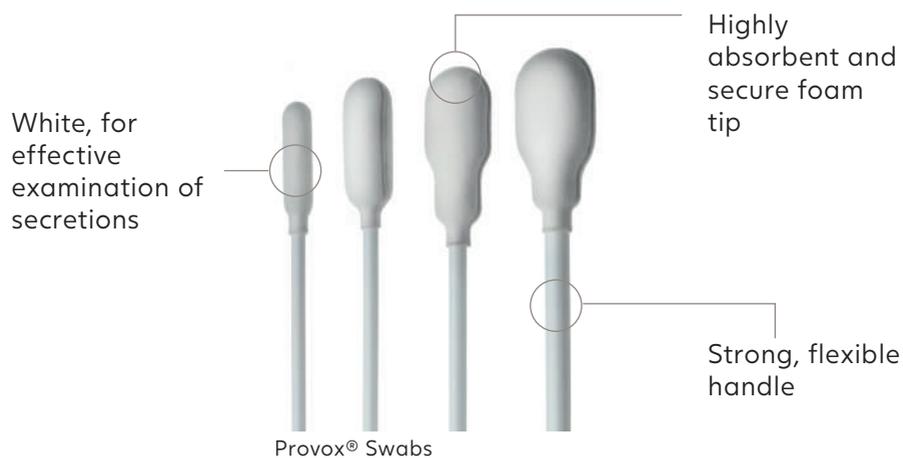

Tube Swabs	For Tube Size	Pack Size	Order Code
Provox Swabs Small	4 - 5mm	50 pcs	8250
Provox Swabs Medium	6 - 8mm	50 pcs	8251
Provox Swabs Large	9 - 10mm	50 pcs	8252
Provox Swab Extra Large	11 - Above	50 pcs	8258
Trachi-Swab Small	4 - 5mm	50 pcs	TRSWB0150
Trachi-Swab Large	5 - 7mm	50 pcs	TRSWB0250
Trachi-Swab Extra Large	8 - Above	50 pcs	TRSWB3050
Insight Tube Swab Small	Up to 6mm	10 pcs	TCS/950
Insight Tube Swab Large	6 - 9mm	10 pcs	TCS/900
SpiroTect Tube Swab		50 pcs	PTA300

Tube Brushes

These brushes are used to clean tracheostomy inner tubes, to help keep them clean and free of mucus or debris.



Tube Brush	Size	Pack Size	Order Code
Provox TubeBrush	8mm	6 pcs	7660
Provox TubeBrush	12mm	6 pcs	7661
Kapitex Tracheo	8mm	2 pcs	TRACC0001
Kapitex Tracheo	10mm	2 pcs	TRACC0002
Kapitex Tracheo	12mm	2 pcs	TRACC0003
Kapitex Tracheo	14mm	2 pcs	TRACC0004

Shower Protectors

Protects the stoma from water when showering.



Cascade

A flexible, waterproof cover which can be worn over the stoma.

Shower Protectors	Pack Size	Order Code
Cascade Shower Protector	2 pcs	AS3835

Stoma Protectors

When worn tightly and completely sealed around the stoma, stoma protectors and covers can help to warm, moisten and filter inhaled air. The Provox Protectors are hand washable up to 3 times. They can also be worn over an HME for additional protection when out during cold weather.

Provox® Protector

- Thick, absorbent foam stoma protector
- Ideal for those with lots of secretions
- Comfortable and discreet
- High quality material with soft-lock mechanism



Provox® Protector Slim

- Lightweight, fabric stoma protector
- Ideal for those with minimal secretions
- Comfortable and discreet
- High quality material with soft-lock mechanism
- Easy to breathe through



Provox® Protector Air

- Light, foam stoma protector
- Ideal for those with thick or sticky secretions - easily wipe-able
- Comfortable and discreet
- High quality material with soft-lock mechanism
- Easy to breathe through



Provox Protectors	White	Blue	Pack Size
Provox Protector Small	7385		10 pcs
Provox Protector Large	7386		10 pcs
Provox Protector Slim Small	7387	7388	10 pcs
Provox Protector Slim Large	7389	7390	10 pcs
Provox Protector Air Small	7391	7392	10 pcs
Provox Protector Air Large	7393	7394	10 pcs



Buchanan Lite
Foam stoma protector



SpiroTect
Foam stoma protector




Hirst
Two layers of knitted net material, no foam, Velcro fastening

Stoma Covers	Colour	Small	Medium	Large	Pack Size	
SpiroTect	Blue	PTA700		PTA702	10 pcs	
Buchanan Lite	White	LABLT2001		LABLT1001	10 pcs	
Buchanan Lite	Beige	LABLT2003		LABLT1003	10 pcs	
Buchanan Lite	Blue	LABLT2002		LABLT1002	10 pcs	
Buchanan	White	BUPO002		BUPO001	10 pcs	
Buchanan Deltanex	White	DNX0001			10 pcs	
Stoma Covers	Beige	White	Black	Blue	Grey	Pack Size
Hirst		BIB				10 pcs

Neckbands


Freevent® Neckband one-piece with twill ties

High quality adjustable neckband with twill ties suitable for use with silicone tubes. Simply cut and adjust to the exact size needed, available in both adult and paediatric sizes.

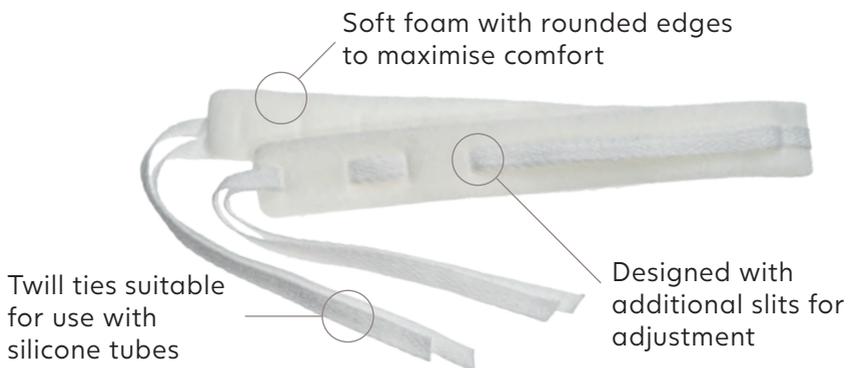

Freevent® Neckband two-piece with VELCRO® brand fastening

High quality adjustable neckband for secure tube positioning. Simple application with VELCRO® fastening, adjustable to adult or paediatric neck size.

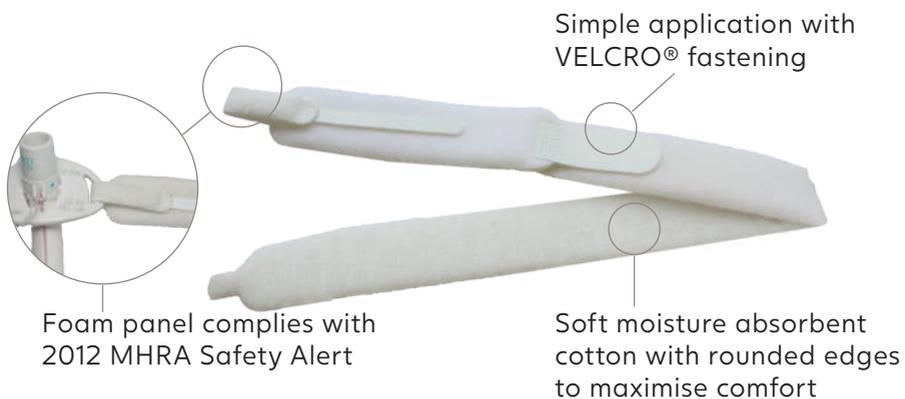

Freevent Neckbands	Adult	Paediatric	Pack Size
Freevent one-piece Freevent one-piece with twill ties	1661	1651	10 pcs
Freevent two-piece with velcro fastening	1662	1652	10 pcs

Neckbands

To help secure a tracheostomy tube in position.



Insight



SpiroTect Two-piece



SpiroTect One-Piece



Trachi-Hold



Marpac One-Piece



Marpac Two-Piece

Tube Holders	Adult	Paediatric	Neo-natal	Pack Size
Insight	TH100	TH150		10 pcs
SpiroTect Two-Piece	PTA201	PTA301		10 pcs
SpiroTect One-Piece	PTA200			10 pcs
Trachi-Hold	TRACC0012	TRACC0014		10 pcs
Marpac One-Piece		107	106	25 pcs
Marpac Two-Piece	201	203MARPAC		10 pcs

Foam Covers

Small foam squares with self-adhesive strip(s), may be worn over the stoma.



Provox®
Tracheofix
2 Strips

Provox®
Tracheofix
1 Strip

Laryngofoam

Purifoam

Foam Protectors	Beige	Ivory	Pack Size
Provox Tracheofix 55mm x 60mm (1 Strip)	1429H		10 pcs
Provox Tracheofix 70mm x 70mm (1 Strip)	1430H		10 pcs
Provox Tracheofix 55mm x 60mm (2 Strips)	1432H		10 pcs
Provox Tracheofix 70mm x 70mm (2 Strips)	1433H		10 pcs
Laryngofoam 51mm x 62mm (1 Strip)	LALFF0003	LALFF0002	30 pcs
Laryngofoam 63mm x 65mm (1 Strip)	LALFF0023		30 pcs
Purifoam 65mm x 6.5mm (1 Strip)		CR3853	30 pcs

Lubricants

Lubricating water soluble jelly to help reduce friction when inserting tubes and buttons. All are sterile until opened.



OptiLube	Size	Pack Size	Order Code
OptiLube Sachet	5g	150	1120
OptiLube Tube	42g	1 pc	1121
OptiLube Tube	82g	1 pc	1122

Skin Wipes and Creams

Skin wipes and creams to help maintain a healthy skin care routine.



Provox® Skin Barrier

- Leaves a protective barrier on the skin to help prevent skin irritation
- Use under tracheostomy dressing
- Convenient portable sachets
- No sting formula, safe for use around the trachea



Provox® Cleaning Towel

- A convenient and hygienic way to clean around the tracheostomy
- Convenient, portable pack size, ideal for handbags, care bags and pockets
- No sting formula



SpiroTect Barrier Wipe



Cavilon Barrier Cream



Cavilon Barrier Film
Pre-soaked swabs

Skin Wipes and Creams	Pack Size	Order Code
Provox Skin Barrier	50 pcs	8011
Provox Cleaning Towel	10 pcs	7244
SpiroTect Barrier Wipe	30 pcs	PTA500
Cavilon Barrier Cream	92 g	3392G
Cavilon Barrier Film	1 ml	3343P

Irrigation Fluid

Saline (salty water) sachets and pods can be used to wash wounds and are suitable to clean around a tracheostoma.



Irripod



Normasol

Irrigation Fluid	Size	Pack Size	Order Code
Irripod	20ml	25 pcs	IRR24N
Normasol	25ml	25 pcs	NOR206B

Gauze Swabs



Spec 28 Non Woven Swabs
Non-woven gauze swabs (squares).

- Gauze swabs have a number of uses for cleaning and caring for the tracheostoma and associated equipment
- Can be used in conjunction with foam swabs
- Highly absorbent and low-linting

Gauze Swab	Size	Pack Size	Order Code
Gauze Swab	100 mm x 100 mm	100 pcs	SWA264T

Sterile Dressing Pack Spec 10
Pre-packed, sterile dressing pack.



Contains:

- Absorbent paper towel (500mm x 450mm) x1
- Gauze swabs (100mm x 100mm; 12ply) x4
- Cotton balls x4, absorbent dressing pad (100 x 200mm) x1
- Sterile field liner

Sterile Dressing Pack Spec 35
Pre-packed, sterile dressing pack.



Contains:

- Absorbent paper towel (500mm x 450mm) x1
- Gauze swabs (100mm x 100mm; 4ply) x4
- Cotton balls x4
- Absorbent dressing pad (100 x 200mm) x1
- Sterile field liner

Dressing Packs	Pack Size	Order Code
Sterile Dressing Pack Spec 10	12 pcs	DP SPEC 10
Sterile Dressing Pack Spec 35	12 pcs	DP SPEC 35

Dressings

Fenestrated dressings have a slit to help them fit around the tracheostomy tube.



ActivHeal

- Highly absorbent foam dressing, suitable for those with lots of secretions
- Fenestrated with non-adhesive backing



Advazorb

- Slimline foam dressing, suitable for those with few secretions
- Fenestrated with non-adhesive backing



Trachi-Dress

- Slimline cotton-type dressing, suitable for those with few secretions
- Fenestrated with non-adhesive backing
- Available in small and large



Trachi-Dress F

- Highly absorbent foam dressing, suitable for those with lots of secretions
- Fenestrated with non-adhesive backing
- Available in small and large

Dressings

Fenestrated dressings have a slit to help them fit around the tracheostomy tube.



Metalline

- Slimline cotton-type dressing, suitable for those with few secretions
- Aluminised backing designed to promote wound healing



Lyofoam Max

- Highly absorbent foam dressing, suitable for those with lots of secretions
- Non-fenestrated with non-adhesive backing
- Available in 6 sizes

Dressings	Size	Pack Size	Order Code
ActivHeal Foam Tracheostomy	100 x 100 mm	10 pcs	10009118
Advazorb (T) Lite (2.5mm thick)	80 x 80 mm	20 pcs	CR4212
Advazorb Plus (T) (5mm thick)	80 x 80 mm	20 pcs	CR3900
Trachi-Dress F Small	60 x 82 mm	10 pcs	TRDRF0001
Trachi-Dress F Large	80 x 100 mm	10 pcs	TRDRF0002
Trachi-Dress Small	60 x 82 mm	10 pcs	TRDRE0001
Trachi-Dress Large	80 x 100 mm	10 pcs	TRDRE0002
Metalline	80 x 90 mm	50 pcs	23 094
Lyofoam Max (603200)	75 x 85 mm	10 pcs	LYO55L
Lyofoam Max (603201)	100 x 100 mm	10 pcs	LYO56H
Lyofoam Max (603203)	100 x 200 mm	10 pcs	LYO57D
Lyofoam Max (603204)	150 x 150 mm	10 pcs	LYO58W
Lyofoam Max (603205)	150 x 200 mm	10 pcs	LYO595
Lyofoam Max (603206)	200 x 200 mm	10 pcs	LYO54P

Jaw Rehabilitation

TheraBite®

For rehabilitation of reduced jaw opening (trismus).

The TheraBite is an easy-to-use device which provides passive stretching of the temporomandibular (jaw) joint (TMJ) to gently and gradually increase the range of motion and maximum interincisal opening (MIO). It's ideal for patients suffering from trismus (jaw hypomobility) who are experiencing difficulty with either biting, chewing, maintaining oral hygiene, speech or swallowing.

The TheraBite® Jaw Motion Rehabilitation System™ is a clinically proven, anatomically correct and portable system designed to restore mobility after trismus and mandibular hypomobility.

The TheraBite® Jaw Motion Rehabilitation System™ can also help to reduce muscle pain by reducing joint inflammation and prevent reduced muscle strength during (chemo)radiation in the head and neck area. The TheraBite Jaw Motion Rehabilitation System is available in both adult and pediatric sizes.

Therabite® System Contains:

- 1 x Jaw Mobiliser
- 4 x Bite Pads
- 1 x Hand Aid
- 30 x Range Of Motion Scales
- 1 x Carry Bag
- 1 x Instructions For Use
- 1 x Patient Progress Log



Therabite	Size	Pack Size	Order Code
Therabite Jaw Motion Rehabilitation System	Adult	1	TH001
Therabite Jaw Motion Rehabilitation System	Paediatric	1	TH002
Therabite Bite Pads	Regular/Adult	4	PA001
Therabite Bite Pads	Edentulous	4	PA002
Therabite Bite Pads	Paediatric	4	PA003



Check with your Healthcare Professional for suitability.



Product shown: TheraBite




“ With the new range of HMEs, I managed to half the number of times I have coughing episodes, which is a blessing.

- Joakim, Sweden

Product shown: Provox Life Home

Contact us

There are several easy ways to contact us:

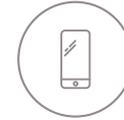

Call or text us

Freephone: 0800 783 1659

Text: 0780 014 0059



Email us

info@atos-care.co.uk



Visit our website

www.atos-care.co.uk



Write to us

Atos Medical, Cartwright House,
Riverside Business Park,
Tottle Road, Nottingham,
NG2 1RT



Product shown: Provox Life Go

Atos

Breathing-Speaking-Living

Atos Care is the distribution arm of Atos Medical, the world-leading supplier of ostomy products for post laryngectomy and tracheotomy patients. Dedicated to making life easier for people living with a neck stoma, Atos Medical has decades of experience, and a deep understanding of what life is like after the operation.

So, although we supply the full range of products, we are much more than just a delivery service. Our expertise and focus mean that we can also provide full professional support and aftercare for you as an individual, wherever you are on your journey. We call this the Atos Circle of Care.

Atos (UK), Cartwright House, Tottle Road,
Nottingham, NG2 1RT
Tel: 0800 783 1659 Text: 0780 014 0059
Email: info.uk@atosmedical.com
Web: www.atos-care.co.uk